## 4 N D **Digital Blood Pressure Monitor**

Model UA-654MR

Instruction Manual Original 1WMPD4004573A

### 1. Dear Customers

- Congratulations on purchasing a state-of-the-art A&D blood pressure monitor. Designed for ease of use and accuracy, this device will facilitate your daily blood pressure regimen.
- We recommend that you read through this manual carefully before ising the device for the first time

# 2. Preliminary Remarks

- The device is designed for use on adults, not newborns or infants
- Environment for use. The device is for use to operate by yourself in the home healthcare environment.
- This device is designed to measure blood pressure and pulse rate of people for diagnosis


### 3. Precautions

- Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- Should be avoided.
   Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
   Avoid tightly folding the cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components.
- The device and cuff are not water resistant. Prevent rain, sweat and
- Water from solling the device and cuff. Measurements may be distorted if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong electrical fields. When reusing the device, confirm that the device is clean
- Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.
   When the AC adapter is used, make sure that the AC adapter can be readily removed from the electrical outlet when necessary.
- Do not modify the device. It may cause accidents or damage to the device
- To measure blood pressure, the arm must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the artery. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time. Take care to avoid accidental strangulation of babies or infants with the
- hose and cable
- Do not wist the air hose during measurement. This may cause injury due to continuous cuff pressure.
   Measuring blood pressure too frequently may cause harm due to blood flow interference. Check that the operation of the device does not result in prolonged impairment of blood circulation, when using the device repeatedly. the device repeatedly.
- If you have had a mastectomy, please consult a doctor before using
- Do not let children use the device by themselves and do not use the device in a place within the reach of infants. It may cause accidents or damage. There are small parts that may cause a choking hazard if swallowed
- Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this blood pressure monitor. Therefore, a minimum distance of 30 cm should be kept from such devices.
- Unplug the AC adapter when not in use during the measurement. Use of accessories not detailed in this manual may compromise safety.
- Should the battery short-circuit, it may become hot and potentially cause burns Allow the device to adapt to the surrounding environment before use
- a haw the back of adapt to the satisfication of the intermediate of the satisfication of the satisfic
- time. That may result in electrical shock
- Do not inflate without wrapping the cuff around the upper arm

#### Contraindications

- The following are precautions for proper use of the device. Do not apply the cuff on an arm with another medical electrical
- equipment attached. The equipment may not function properly. People who have a severe circulatory deficit in the arm must consult a doctor before using the device, to avoid medical problems.
- Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatmer
- Do not apply the cuff on an arm with an unhealed wound. Do not apply the cuff on an arm receiving an intravenous drip or blood
- transfusion. It may cause injury or accidents.
- Do not use the device where flammable gases such as anesthetic gases are present. It may cause an explosion.
   Do not use the device in highly concentrated oxygen environments,
- such as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire or explosion.

# 4. Parts Identification

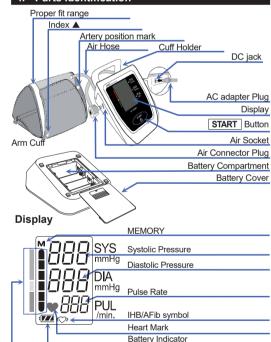

#### Symbols printed on the device case.

Symbols	Function / Meaning
U U	Standby and Turn the device on.
SYS	Systolic blood pressure in mmHg
DIA	Diastolic blood pressure in mmHg
PUL	Pulse per minute
$\oplus \square \ominus$	Battery installation guide
	Direct current
Ŕ	Type BF: Device, cuff and tubing are designed to provide special protection against electrical shocks.
	Manufacturer
2020 ლ	Date of manufacture
X	Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.
SN	Serial number
6	Refer to instruction manual/booklet
⊖-€-⊕	Polarity of DC jack
IP	International protection symbol
Ť	Keep dry

## Using the Monito

- 6.1. Installing / Changing The Batteries
- 1. Remove the battery cover
- 2. Remove the used batteries from the battery compartment in case of changing them. 3. Insert new batteries into the battery compartment as shown, taking care that
- the polarities (+) and (-) are correct. 4. Replace the battery cover Use only R6P, LR6 or AA batteries
- CAUTION
  - Insert the batteries as shown in the battery compartment. If installed incorrectly, the device will not work.

Ster

- When display and the device announces that the battery needs to be replaced, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction. Replace the batteries two seconds or more after the device turns off.
- does not appear when the batteries are drained
- The battery life varies with the ambient temperature and may be shorter at low temperatures. Generally, four new R6P batteries will last approximately for three months when used twice for measurement each day.
- Use the specified batteries only. The batteries provided with the device are for testing the device performance and may have a limited life.
   Remove the batteries if the device is not to be used for a long time.

#### The batteries may leak and cause a malfunction 6.2. Installing the Cuff Holder

This device comes equipped with a removable cuff holder for your convenience. When not in use, simply slip your cuff into the holder for storage.


- To install the cuff holder Align the cuff holder with the rear of the device
- Slide the cuff holder into the rear of the device until it snaps securely into place.
- To remove the cuff holder 1. Push the tab on the cuff holder away from the blood pressure monitor
- 2. Slide the cuff holder away from the blood pressure monitor as shown to the right

# 6.3. Connecting The Air Hose

Insert the air connector plug into the air socket firmly. Air Hose

6.4. Connecting The

**AC Adapter** Air Socket

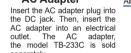

separately. When disconnecting the AC

- adapter from the electrical outlet, grasp and pull the AC adapter body out of the outlet.
- When disconnecting the AC adapter plug from the blood pressure monitor, grasp and pull the AC adapter plug out of the monitor.

# 6.5. Selecting The Correct Cuff

Using the correct cuff size is important for an accurate reading. If the cuff is not the proper size, the reading may yield an incorrect blood pressure value.

Air Connector Plug

AC adapter Plug

DC Jack

- The arm size is printed on each cuff. The index ▲ and proper fit range, on the cuff, tell you if you are applying the correct cuff. Refer to "6.6 Applying The Arm Cuff".
- □ If the index ▲ points outside of the range, contact your local dealer to
- ourchase a replacement cuff. The arm cuff is a consumable. If it becomes worn, purchase a new one

Arm Size	Recommended Cuff Size	Catalog Number
31 cm to 45 cm	Large Adult cuff	CUF-D-LA-ISO
22 cm to 42 cm	Wide range cuff	CUF-I-ISO
23 cm to 37 cm	Semi Large cuff	CUF-D-MA-ISO
22 cm to 32 cm	Adult cuff	CUF-F-A-ISO

Model UA-654MR is not designed for using a small cuff. Note Symbols printed on the cuff.

# S

ymbols printed on the curr.			
Symbols	Function / Meaning	Recommended Action	
•	Artery Position Mark	Set the ● mark on the artery of the upper arm or in line with the ring finger on the inside of the arm.	
	Index		
REF	Catalog number		
А	Proper fit range for the adult cuff. It's printed on the adult cuff.		
	Over range printed on the semi large adult cuff.	Use the large adult cuff instead of the semi large adult cuff.	
L	Proper fit range for the large adult cuff. It's printed on the large adult cuff.		
$\langle S \rangle$	Under range printed on the semi large adult cuff.		
$\langle M \rangle$	Under range printed on the large adult cuff.	Use the semi large adult cuff instead of the large adult cuff.	
LOT	Lot number		
	Manufacture		
MD	Medical Device		

<b>7.1. Normal Measurem</b> 1. Place the cuff on the arm at	ent At heart level«
heart level (preferably the left arm). Sit quietly during measurement.	Press the START button
<ol> <li>Press the START button. All of the display segments are displayed. Zero is displayed</li> </ol>	All of the display segments
blinking briefly. Then the display changes, as indicated in the figure at the right, as the measurement begins. The cuff starts to inflate. It is normal for the cuff to feel very tight. A pressure bar indicator is	Zero display
displayed, as in the figure at the right, during inflation.	Starts inflation
Note: If you wish to stop inflation at any time, press the [START] button again. 3. When inflation is complete, deflation starts automatically	Pressurizing
and the ♥ (heart mark) blinks, indicating that the measurement is in progress. Once the pulse is detected, the mark blinks with each pulse beat.	Measurement in progress
Note: If an appropriate pressure	Diastolic pressure
is not obtained, the device starts to inflate again automatically.	WHO classification
<ol> <li>When the measurement is</li> </ol>	Pulse rate
complete, the systolic and diastolic pressure readings and pulse rate are displayed. The	Exhausts remaining air automatically
cuff exhausts the remaining air and deflates completely.	Turn off with the

7. Measurements

12. About Blood Pressure

What is Blood Pressure?

measured in millimeters of mercury (mmHg)

13. WHO Classification Indicator

Possible Reason

Battery terminals are not in the corre

Battery voltage is too low. t blinks

If the batteries are drained completely

the mark does not

The cuff is not

applied properly

You moved your

neasurement

not correct.

The value is

different from that

measured at a clinic or doctor's office.

arm or body during

The cuff position is

Batteries are

drained.

osition.

appear.

Each segment of the bar indicator

corresponds to the WHO blood

: The indicator displays a segment,

based on the current data

corresponding to the WHO

pressure classification

14. Troubleshooting

classification.

Problem

appears on

the display, even when the power is

The cuff does

not inflate.

The device

does not measure. Readings are

too high or too

Other

Туре

15. Maintenance

authorized dealers.

16. Technical Data

Measurement method

Measurement range

Measurement

Power supply

Classification

Clinical test

EMD

Memory

Weight

No data

36 75

14

пο.

Dimensions

Applied part

TB-233C

Useful life

Operating conditions

Ingress protection

Accessory AC adapter

Transport / Storage conditions

Number of measurem

accuracy

Rating

turned on.

Nothing

····

Ü

÷

68

Ŧ

ISO

90

। १३५१ १८०

Ċ

Ü

Refer to the section

Blood pressure is the force exerted by blood against the walls of

the arteries. Systolic pressure occurs when the heart contracts.

Diastolic pressure occurs when the heart expands. Blood pressure is

ones.

nes

WHO Classification Indicator

Mild hypertension

High normal

Normal

Optimal

Recommended Action Replace all batteries with new

Reinstall the batteries with negative and positive terminals matching those indicated on the

Replace all batteries with new

Make sure you remain still and

Sit comfortably and still. Place

your arm on a table with your palm facing upward and the cuff at the same level as your heart.

quiet during measurement.

If you have a very weak or

our blood pressure At a clinic or doctor's office

he doctor's readings.

measurement

Note: If the actions described above do not solve the problem, contact

attempt to do so will make your warranty invalid.

UA-645MR

Pressure:

Pulse:

Pulse:

holder

Device

Device

AC adapter:

The adapter is to connect the device to a power source at home. Please contact your local A&D dealer for purchasing. The AC adapter

For indoor use only

Fuse EC directive device label

EAC certification device label Polarity of AC adapter plug

Class II device

Thermal fuse

Please contact your local A&D dealer for purchasing. The AC adapter is required to be inspected or replaced

Function / Meaning

Cuff

Cuff:

is required to be inspected or replaced periodically

periodically

Symbols

G

<u>C€</u> [Ⅲ] ⊖<del>©</del>⊕

Symbols that are printed on the AC adapter.

Pressure

DC 6V 3W

Systolic pressure:

the dealer. Do not attempt to open or repair this product, as any

Do not open the device. It uses delicate electrical components and an

intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact the authorized dealer in your area or our customer service department. The A&D customer service will provide technical information, spare parts and units to

The device was designed and manufactured for a long service life. However it is generally recommended to have the device inspected every 2 years, to ensure proper functioning and accuracy. Please contact the authorized dealer in your area or A&D for maintenance.

Oscillometric measurement

Diastolic pressure: 40 - 200 mmHg

4 x 1.5V batteries (R6P, LR6 or AA) or

50 / 60 Hz 0.15A)) (Not included)

According to ISO81060-2: 2013

0 - 299 mmHa

±3 mmHg

±5 %

AC adapter (TB-233C INPUT: AC 100 - 240V

Approx. 700 times LR6 (alkaline batteries) Approx. 200 times R6P (manganese batteries) With pressure value 180 mmHg, room temperature 23 °C.

Internally powered ME equipment(Supplied by batteries) / Class II (Supplied by adapter) Continuous operation mode

In the clinical validation study, K5 was used on 85 subjects for determination of diastolic blood pressure. IEC 60601-1-2: 2014

Last 60 measurements +10 to +40 °C / 15 to 85 %RH / 800 to 1060 hPa -20 to +60 °C / 10 to 95 %RH / 700 to 1060 hPa

Approx. 106 [W] x 67 [H] x 143 [D] mm excluding the cuff holder

Approx. 240 g, excluding the batteries and cuff

IP20

Type BF 🖈

5 years (when used six times a day) 2 years (when used six times a day)

5 years (when used six times a day)

60 - 279 mmHg

40 - 180 beats / minute

irregular heartbeat, the device

an apprehension may cause

may have difficulty in determining

an elevated reading. Home measurement reduces the effect of outside influences on blood

pressure readings, supplements

Remove the batteries. Place ther back properly and take another

battery compartment.

Apply the cuff correctly

Severe hypertension

Moderate hypertension

- and deflates completely 5. Press the START button
- again to turn off the power
- Note: Model UA-654MR is provided with an automatic power shut-off function. Allow at least 3 minutes between measurements on the same person
- 7.2. Measurement with the Desired Systolic Pressure
- If re-inflation occurs repeatedly use the following methods At heart level — 🐢 🔤 If your systolic pressure is expected to exceed 230 mmHg, Press the START button 🛈= use this procedure.
- Place the cuff on the arm at heart level (preferably the left arm).
- 2. Press the START button. 3. During the zero blinks, press and hold the <u>START</u> button until a number about 30 to
- Press and hold the START button 40 mmHa higher than your Inflation in progress expected systolic pressure appears. Release the START 4. When the desired number is reached, release the START button to start measurement The desired pressure
- Then continue to measure your blood pressure as described on the section "7.1 Normal Measurement "7.1 Normal Measurement"

#### 7.3. Notes for Accurate Measurement

- Sit comfortably on a chair. Rest your arm on the table. Do not cross your legs. Keep your feet on the floor and straighten your back.
   Sit down in a comfortable position. Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.
   Place the center of the cuff at the same level as your heart.
- Relax for about five to ten minutes before measurement. If you are excited or depressed by emotional stress, the measurement will reflect this stress as a higher (or lower) than normal blood pressure
- reading and the pulse reading will usually be faster than normal
- Remain still and keep quiet during measurement. Do not measure immediately after physical exercise or a bath. Rest for twenty or thirty minutes before taking the measurement. An individual's blood pressure varies constantly, depending on what
- you are doing and what you have eaten. What you drink can have a very strong and rapid effect on your blood pressure. This device bases its measurements on the heartheat. If you have a very weak or irregular heartbeat, the device may have difficulty determining vour blood pressure.
- Should the device detect a condition that is abnormal, it will stop the measurement and display an error symbol. Refer to the section "5. Symbols" for the description of the symbols.
- This device is intended for use by adults. Consult with your physician before
- using this device on a child. A child should not use this device unattended Try to measure your blood pressure at the same time every day
- The automatic blood pressure monitor's performance may be affected by excessive temperature or humidity, or altitude.

## Recalling the Memory Data

8.1. Recalling Data

displayed, press and hold the <u>START</u> button to recall the stored data.

1. When nothing is

2. Release the button

average data.

3. The data number

measurement

when displaying the

and stored data are

in order from the last

4. The display will turn off

data is displayed.

Note: If you press the START button while

device turns off.

8.2. Clearing Data

recalling data, the

automatically after all

automatically displayed

The device automatically stores up to sixty blood pressure and pulse measurements in memory. Data stored in memory are assigned a data number in the order of the newest to the oldest. The oldest data displays as  $r_{10}$ , it. The **M** symbol in the upper left corner of the display indicates that you are viewing previous data stored in memory.

Press and hold the

START button


M mark

Average Data

Data number

Stored Data


Turns off automatically

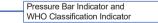

### Symbols

### Symbols that appear on the display


Symbols	Function / Meaning	Recommended Action	
٠	Appears while measurement is in progress. It blinks when the pulse is detected.	Measurement is in progress. Remain as still as possible.	
(())	Appears when an irregular heartbeat is detected. It may light when a very slight vibration like shivering or shaking is detected.		
M Previous measurements stored in memory.			
[]	FULL BATTERY The battery power indicator during measurement.		
C 📕	LOW BATTERY The battery power is low when it blinks.	Replace all batteries with new ones when the mark blinks.	
	Unstable blood pressure due to movement during measurement.	Take another measurement. Remain still during measurement.	
Err	The systolic and diastolic values are within 10 mmHg of each other. The pressure value did not increase during the inflation.	Apply the cuff correctly, and	
ÊŰF	The cuff is not applied correctly.	take another measurement.	
Ε	PUL DISPLAY ERROR The pulse is not detected correctly.		
Êrê		Remove the batteries and press the START button, and then install the	
Err	Device internal error	batteries again. If the error still appears, contact the dealer.	

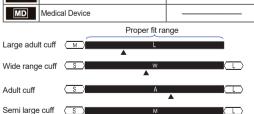

### 6.6. Applying The Arm Cuff

- 1. Wrap the cuff around the upper arm, about 1 to 2 cm above the inside of the elbow, as shown
- Place the cuff directly against the skin, as clothing may cause a faint pulse and result in a measurement error.
- 2. Constriction of the upper arm, caused by rolling up a shirtsleeve may prevent accurate readings.
- 3. Confirm that the index **A** points within the proper fit range.
- Note: During measurement, it is normal for the cuff to feel very tight. (Do not be alarmed).



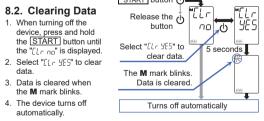

Press and hold the

START button 🛈

#### 9. What is the I.H.B./AFib

When the monitor detects an irregular rhythm during the measurements, the I.H.B./AFib indicator will appear on the display with the measurement values


Note: We recommend contacting your physician if you see this (O) symbol frequently.

### 10. What is AFib

The heart contracts due to electrical signals occurring in heart and sends blood through the body. Atrial fibrillation (AFib) occurs when the electrical signal in the atrium becomes confused and leads to disturbances in the pulse interval. AFib can cause blood to stagnate in the heart, which can asily create clots of blood, a cause of stroke and heart attack

### 11. Pressure Bar Indicator

The indicator monitors the progress of pressure during measurement

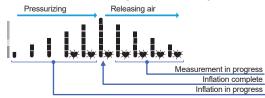

Acces	sories sold separate	ly	
	-	-	
Cuff	Catalog Number	Cuff Size	Arm Size
	CUF-D-LA-ISO	Large Adult cuff	31 cm to 45 cm
	CUF-I-ISO	Wide range cuff	22 cm to 42 cm
	CUF-D-MA-ISO		23 cm to 37 cm
	CUF-F-A-ISO	Adult cuff	22 cm to 32 cm
Arm size: The circumference at the biceps.			
			-

AC Adapter	Catalog Number	Plug (Outlet type)
	TB-233C	Туре С

Note: Specifications are subject to change without prior notice IP classification is the degrees of protection provided by enclosures in accordance with IEC 60529. This device is protected against solid foreign objects of 12 mm diameter and greater such as a finger. This device is not protected against water