# UA-621

## Upper Arm Blood Pressure Monitor

OPERATION MANUAL (English)  
Manuel d'utilisation (French)  
Manual de funcionamiento (Spanish)  
Manual de Operações (Portuguese)  
Manuale d'uso (Italian)  
Bedienungsanleitung (German)  
Kullanım Kilavuzu (Turkey)  
دليل التشغيل (Arabic)  
ഓപ്പറേഷൻ മാനുവൽ (Malayalam)  
操作手冊 (Chinese)

**AND**  
A&D Medical

**Thank you!**

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**Grazie!**

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**Gracias!**

**Obrigado!**

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Thank you for purchasing an A&D Monitor!

## Index

1. Getting Started	
- Set Components	2
- Device Description	2
- Installing the batteries	3
2. Measurement	
- Securing the Cuff	3
- Measuring your Blood Pressure	4
- Memory Recall	4
- Tips for accurate Measurements	5


# Getting Started

## 1. Set Components


This package contains the following items:

- Blood Pressure Monitor
- Arm Cuff with air tube
- Batteries (AA) x 4
- Soft Carrying Case
- Instruction Manual
- Operation Manual

## 2. Device Description

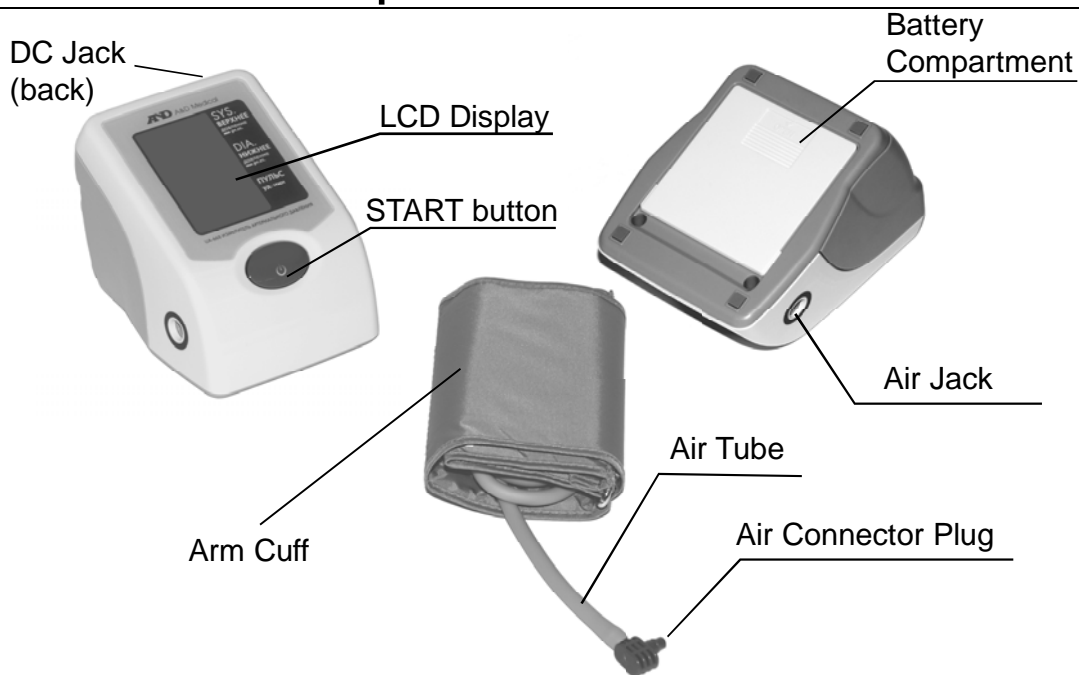### Display Description
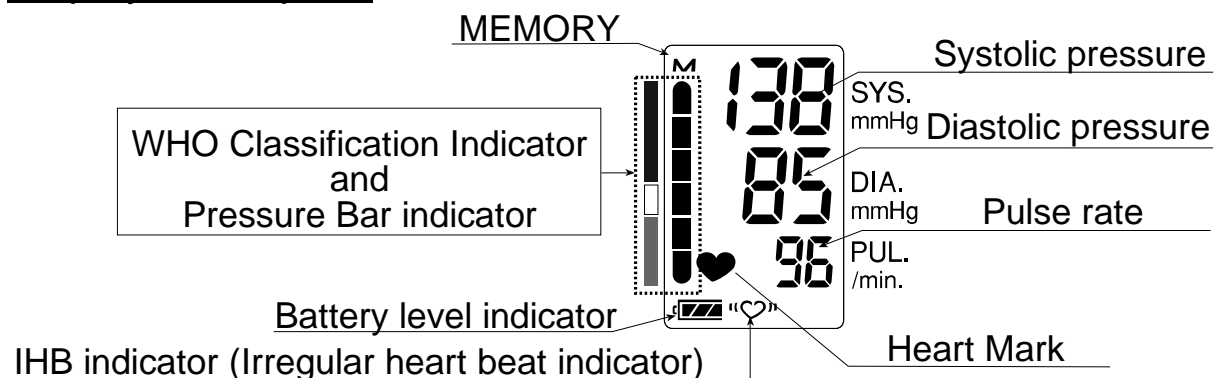
## 3. Installing the Batteries

1. Open the battery cover by sliding it upward.
2. Install or replace batteries in the battery compartment as shown. Please check that the polarities (+) and (-) are correct.
3. Close the cover by sliding the battery cover down.  
Use only R6P, LR6 or AA batteries.

## Measurement

### 1. Securing the Cuff

1. Insert the air connector plug into the air jack on the left side of the monitor.
2. Put your left arm into the cuff loop--the air tube should be on the bottom--and wrap the cuff around your upper arm


Where:

The cuff should be about 2 - 3 cm above the elbow. Place the cuff directly against the skin, as clothing may cause a faint pulse and result in a measurement error.

Note:

- Please make sure that the cuff is wrapped securely around the arm, but not too tightly. Adjust cuff tightness by putting 1 or 2 fingers between the cuff and arm.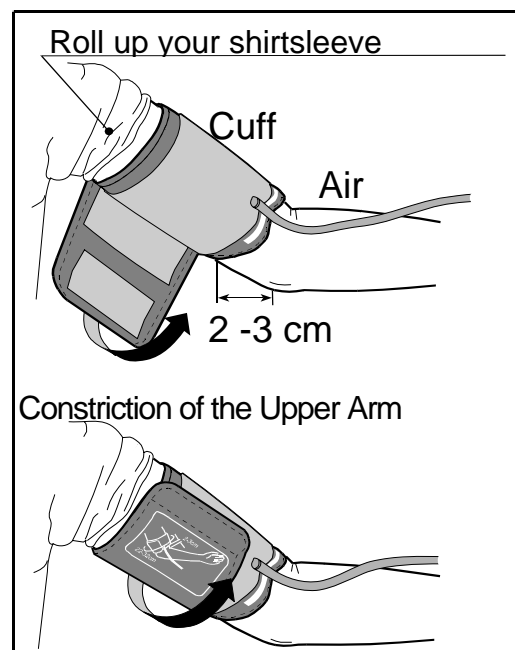

## 2. Measuring your blood pressure

1. Sit on a chair with you feet flat on the floor. Put your arm on a table and relax your posture. The cuff should be about the same level as your heart.
2. Press the START button to start the measurement.
3. Keep quiet and still until the measurement is complete.

## 3. Memory Recall

This blood pressure monitor memorizes measurement data (Systolic, Diastolic and Pulse value) automatically after each measurement is complete.

- Total No. of memory: 30 sets
- When the memory reaches maximum capacity, it deletes the oldest memory data.
- All the memory data will be erased when the batteries are removed from the monitor.

1. Press the START button and keep the button depressed for approx. 3 seconds until the “n01” mark appears on the display.
2. Stop depressing the button.
3. All the data shall be displayed automatically. (Newest data will be displayed first)

## 4. Tips for accurate Measurements

### Before measurement

- 1) Measure blood pressure at the same time each day; blood pressure fluctuates throughout the day.
- 1) Relax for five to ten minutes before taking your measurement.
- 2) Sit in a relaxed position.
- 3) Properly secure the cuff on your arm as described above.

### During the measurement

- 1) Do not talk.
- 2) Keep quiet and stay calm for the entire process.
- 3) Keep the cuff at heart level.

### After the measurement

- 1) Do not become alarmed after each measurement result.
- 2) Consult your physician and avoid self-diagnosis.

### Several factors that increase your blood pressure

Quick movement

Sudden change in ambient temperature

Excitement

Alcohol or smoking

Obesity

Lack of sleep

**For more details, please read the instruction manual included in the package.**

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**A&D Company, Limited**