



Wrist Digital Blood Pressure Monitor

Model UB-533

Instruction Manual
Original

ENGLISH

Manuel d'instructions
Traduction

FRANÇAIS

Manual de Instrucciones
Traducción

ESPAÑOL

Manuale di Istruzioni
Traduzione

ITALIANO

使用手冊
翻譯

中文



1WMPD4005018A
Issued Date: 19 Aug. 2025

Contents

Dear Customers.....	2
Preliminary Remarks	2
Intended purpose	2
Clinical Benefit.....	2
Precautions	3
Parts Identification	5
Symbols	6
Using the Monitor	9
1. Inserting / Changing the Batteries	9
2. Selecting a User.....	10
3. Adjusting the Built-in Clock Before Use.....	10
4. Applying the Cuff.....	11
5. How to Take Accurate Measurements	11
6. The C.P.G. Function Switch.....	12
7. Selecting the C.P.G. and my C.P.G.....	12
8. Measurement	13
9. After Measurement	13
Measurements	14
Recalling the Memory Data.....	15
Deleting Data Stored in Memory	16
What Is The IHB/AFib Indicator?	16
What Is The AFib?	16
% IHB/AFib.....	17
Pressure Bar Indicator.....	18
WHO Classification Indicator.....	18
The C.P.G. Indicator.....	19
About Blood Pressure	20
What is Blood Pressure?.....	20
What is Hypertension and How is it Controlled?	20
Why Measure Blood Pressure at Home?	20
WHO Blood Pressure Classification.....	21
Blood Pressure Variations.....	21
Troubleshooting	22
Maintenance	23
Technical Data.....	23

Dear Customers

Thank you for purchasing a state-of-the-art A&D blood pressure monitor, one of the most advanced monitors available today. Designed for ease of use and accuracy, this device will facilitate your daily blood pressure regimen.

We recommend that you read through this manual carefully before using the device for the first time.

Preliminary Remarks

- ❑ This device conforms to the Medical Device Regulations (EU 2017/745) and UKCA Medical Device Regulations 2002 for Medical Products.
This is made evident by the CE2797 and UKCA0086 mark of conformity. (2797 and 0086: The reference numbers to the involved notified body.)
- ❑ The device is designed for use by people 13 years and older, not newborns or infants.
- ❑ This device fulfils the following provisions.



Medical Device Regulations (EU 2017/745)



UKCA Medical Devices Regulations 2002

Intended purpose

The UB-533 Digital Blood Pressure Monitor is intended to be used by patients to measure systolic and diastolic blood pressure and pulse rate in the home environment. The UB-533 provides the patient with an indication of an irregular heartbeat allowing further medical attention to be sort.

Clinical Benefit

Successful assessment of blood pressure reading in accordance with the device's intended purpose.

Precautions

- ❑ Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- ❑ Clean the device with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzine, thinner or other harsh chemicals to clean the device.
- ❑ Avoid tightly folding the cuff for long periods, as such treatment may shorten the life of the components.
- ❑ The device is not water resistant. Prevent rain, sweat and water from damaging the device.
- ❑ Measurements may be distorted if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong electrical fields.
- ❑ Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.
- ❑ When reusing the device, confirm that the device is clean.
- ❑ Do not modify the device. It may cause accidents or damage to the device.
- ❑ To measure blood pressure, the wrist must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the wrist. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time.
- ❑ Wireless communication device, such as home networking devices, mobile phones, cordless phones and their base stations, and walkie-talkies can affect this blood pressure monitor. Therefore, a minimum distance of 30 cm should be kept from such devices.
- ❑ Measuring blood pressure too frequently may cause harm due to blood flow interference. Check that the operation of the device does not result in prolonged impairment of blood circulation, when using the device repeatedly.
- ❑ Clinical testing has not been conducted on newborn infants and pregnant women. Do not use on newborn infants or pregnant women.
- ❑ If you have had a mastectomy or lymph node clearance, please consult a doctor before using the device.
- ❑ Do not let children use the device by themselves and do not use the device in a place within the reach of infants. This may cause accidents or damage.
- ❑ There are small parts that may cause a choking hazard if swallowed by mistake by infants.
- ❑ Do not touch the batteries and the patient at the same time. That may result in electrical shock.
- ❑ In the case of single components failure enclosure of near cuff may become hot and potentially cause malfunction. (Maximum 46°C)



- ❑ Use of accessories not detailed in this manual may compromise safety.
- ❑ Should the battery short-circuit, it may become hot and potentially cause burns.
- ❑ Allow the device to adapt to the surrounding environment before use (about one hour).
- ❑ Do not inflate without wrapping the cuff around your wrist.
- ❑ This is a medical device for use by layperson. Please consult your healthcare provider with any questions or concerns you may have regarding your condition.
- ❑ When any serious incident occurs in relation to this device, report to its manufacturer and the competent authority in your country.
- ❑ Confirm for proper operation before use, if the packaging is damaged, unintentionally opened and exposed to environmental conditions outside of those specified.
- ❑ This device is not intended to diagnose heart arrhythmias. If the Irregular Heart Beat indicator illuminates frequently and is unrelated to patient movement during blood pressure measurement, further medical attention must be sort.

Contraindications

The following are precautions for proper use of the device.


- ❑ Do not apply the device to a wrist with other medical electrical equipment attached. The equipment may not function properly.
- ❑ People who have a severe circulatory deficit in the arm must consult a doctor before using the device to avoid medical problems.
- ❑ Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatment.
- ❑ Do not apply the device on a wrist with an unhealed wound.
- ❑ Do not apply the device on an arm receiving an intravenous drip or blood transfusion. It may cause injury or accidents.
- ❑ Do not use the device where flammable gases such as anesthetic gases are present. It may cause an explosion.
- ❑ Do not use the device in highly concentrated oxygen environments, such as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire or explosion.
- ❑ Do not provide any servicing and perform maintenance while the medical device is in use.

Parts Identification

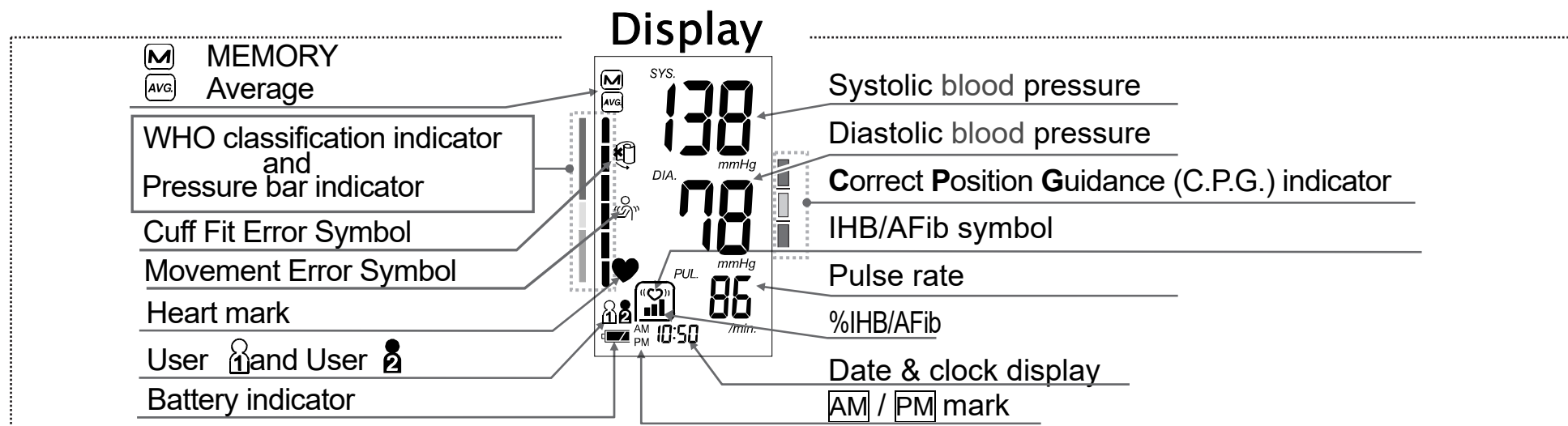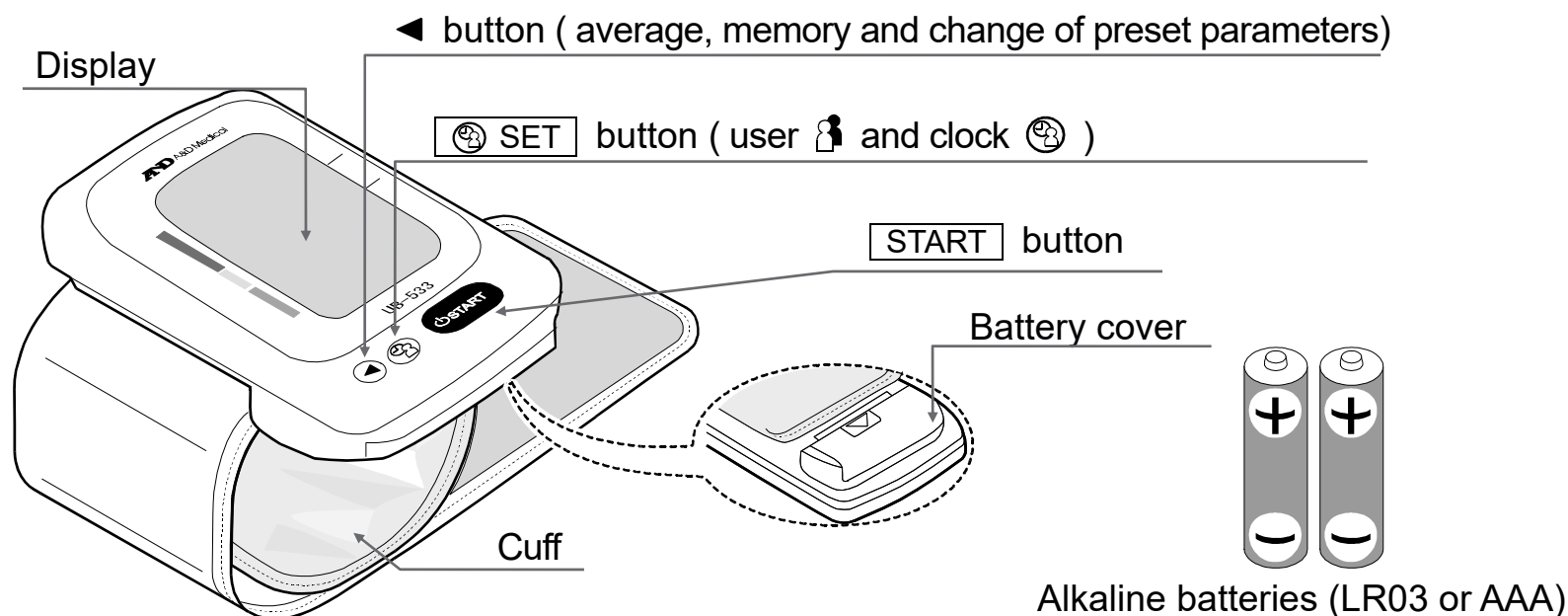

Symbols

The symbols that are printed on the device and the packaging.

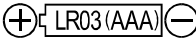Symbols	Function / Meaning
	Press  to turn on the device. Press  during measurement to enter standby.
	Battery installation guide
	Direct current
	Serial number
	Unique Device Identifier
2023 	Date of manufacture
	Type BF applied part
	CE marking
	European Authorized Representative
IP	International protection symbol
	WEEE label
	Manufacturer
	UK Conformity Assessed marking
	UK Responsible Person
	Refer to instruction manual / booklet *1

Symbols	Function / Meaning
	Keep dry
	Medical Device
	Temperature limit
	Humidity limitation
	Atmospheric pressure limitation
	EU Importer
	UK Importer

*1 The symbol color: Blue

Symbols that appear on the display

Symbols	Function / Meaning / Recommended Action
	Appears while measurement is in progress. It blinks when the pulse is detected. Remain as still as possible.
	IHB/AFib symbol appears when an irregular heartbeat is detected. It may light when a very slight vibration like shivering or shaking is detected. Apply the cuff correctly, then take another measurement. If the «  » symbol continues to appear, we recommend you to consult with your physician.
	Appears when a body or arm movement is detected. The reading may yield an incorrect value. Take another measurement. Remain still during the measurement.
	 appears during the measurement when the cuff is attached loosely. The reading may yield an incorrect value. Apply the cuff correctly, and take another measurement.
 	User 1 and user 2

Symbols	Function / Meaning	Recommended Action
	Deleted rate of IHB/AFib in memory $\% \text{IHB/AFib} = \frac{\text{Number of detected IHB/AFibs in memory}}{\text{Total number}} \times 100 \%$	_____
	Displayed when the data in memory is displayed.	_____
	Average data	_____
	FULL BATTERY The battery power indicator during the measurement	_____
	LOW BATTERY The battery is low when it blinks	Replace all batteries with new ones when the indicator blinks.
ϵ_1 or ϵ_2	Unstable blood pressure due to movement during the measurement	Take another measurement. Remain still during the measurement.
	The systolic and diastolic values are within 10 mmHg from each other.	
ϵ_3	The pressure value did not increase during inflation.	Apply the cuff correctly, and take another measurement.
	The cuff is not applied correctly.	
ϵ	PUL. DISPLAY ERROR The pulse is not detected correctly.	
ϵ_{ϵ}	Blood pressure monitor internal error	Remove the batteries and press the START button, and then install the batteries again. If the error still appears, contact the dealer.
ϵ_g		
SYS.	Systolic blood pressure in mmHg	_____
DIA.	Diastolic blood pressure in mmHg	_____
PUL	Pulse per minute	_____
AM	Data taken between 4:00 and 9:59	_____
PM	Data taken between 18:00 and 1:59	_____


Using the Monitor

1. Inserting / Changing the Batteries


1. Remove the battery cover.
2. Remove the used batteries and insert new batteries into the battery compartment as shown, making sure that the polarities (+ and -) are correct. Use only LR03 or AAA batteries.
3. Putting on the battery cover.


⚠CAUTIONS

- ❑ Insert the batteries as shown in the battery compartment. If inserted incorrectly, the device will not work.
- ❑ When  (LOW BATTERY mark) blinks on the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- ❑  (LOW BATTERY mark) does not appear when the batteries are drained.
- ❑ The battery life varies with the ambient temperature and may become shorter at low temperatures. Generally, two new LR03/AAA batteries will last approximately for three months when used twice for measurement each day.
- ❑ Use the specified batteries only. The batteries provided with the device are for testing and may have a limited life.
- ❑ Remove the batteries if the device is not to be used for a long period of time. The batteries may cause leakage and operational issues.
- ❑ When removing the batteries, preset parameters (of clock, user and my C.P.G.) are reset.





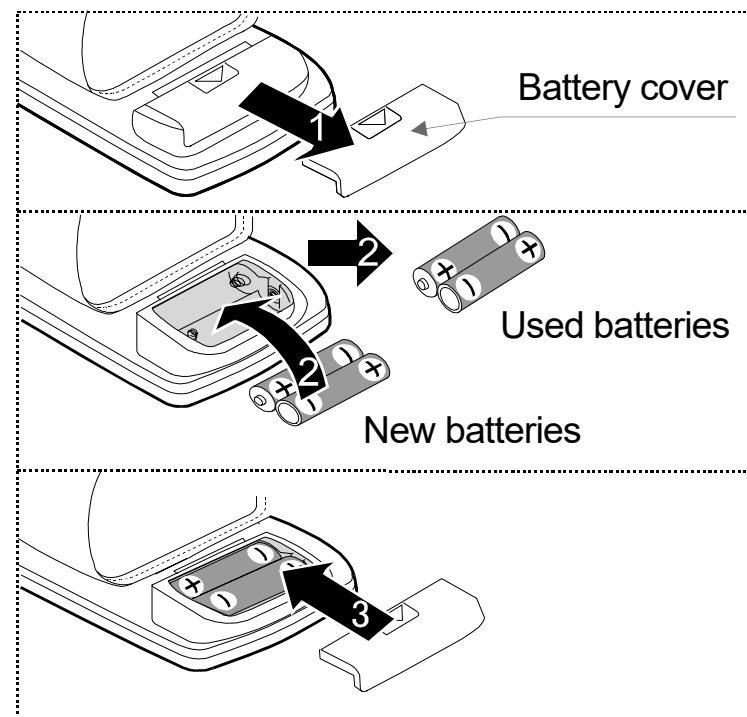



2. Selecting a User

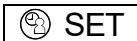1. Press the  SET button when turning off the device. The indicator  or  is blinking.
2. Select a user from user  and user  using the  SET button.
Press the  START button to turn off the device.
After three minutes of non-operation, the device will be turned off automatically.

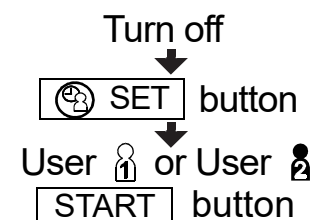

3. Adjusting the Built-in Clock Before Use

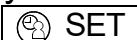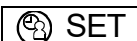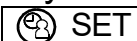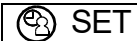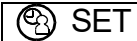1. Press and hold the  SET button until the year starts blinking.
2. Select the year using the  button.
Press the  SET button to set the current year and move to month/day selection.
The date can be set anywhere between the years 2010 and 2059.
3. Select the month using the  button.
Press the  SET button to set the current month and move to day selection.
4. Select the day using the  button.
Press the  SET button to set the current day and move to hour/minute selection.
5. Select the hour using the  button.
Press the  SET button to set the current hour and move to minute selection.
6. Select the minute using the  button.
Press the  SET button to proceed to "6.The C.P.G.Function Switch".

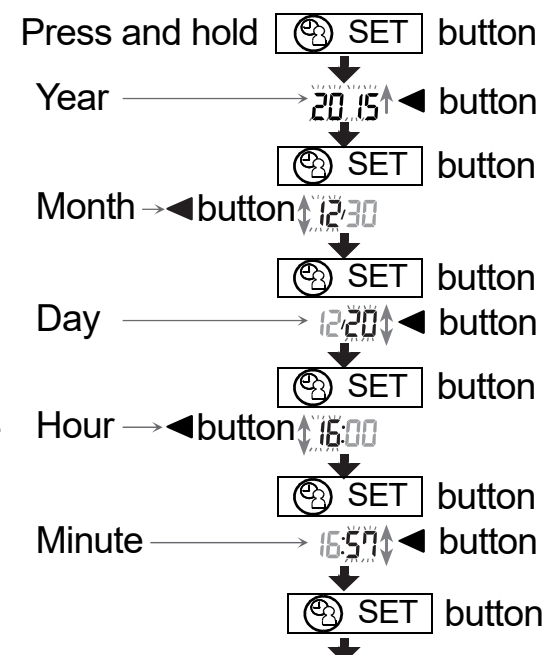

The operation proceeds to
"6. The C.P.G. Function Switch".


- ❑ Holding down the  button will change the value continuously.
- ❑ Pressing the  START button will turn the device off anytime.

Notes: After three minutes of non-operation, the device will be turned off automatically. When the clock has not been set,  is indicated for the clock display. When removing the batteries, preset parameters (of clock, user and my C.P.G.) are reset.

4. Applying the Cuff

1. Wrap the cuff around your wrist about 1 cm above your hand as shown in the figure at the right.
2. Apply the cuff tightly using the Velcro strip.

Note: For accurate measurements, apply the cuff tightly and measure on a bare wrist.

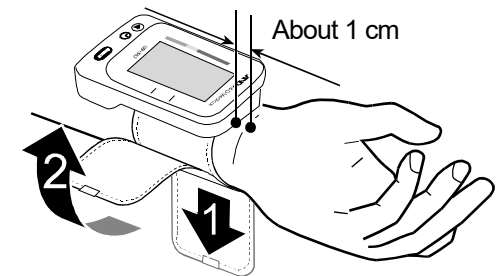


5. How to Take Accurate Measurements


For the most accurate blood pressure measurement:



- ☐ Remain still and keep quiet during the measurement.
- ☐ Sit down in a comfortable position. Place your elbow on a table with your palm facing upward and the cuff is at the same level as your heart.
- ☐ Relax for about five to ten minutes before taking the measurement. If you are excited or depressed by emotional stress, the blood pressure reading may be higher (or lower) than a normal blood pressure reading and the pulse rate will usually be faster than normal.
- ☐ Try to measure your blood pressure at about the same time every day.
- ☐ An individual's blood pressure varies constantly, depending on what you are doing and what you have eaten. What you drink may have a strong and rapid effect on your blood pressure.
- ☐ Do not measure immediately after physical exercise or a bath. Rest for twenty or thirty minutes before taking the measurement.
- ☐ Do not cross your legs. Keep your feet flat on the floor and straighten your back.
- ☐ This device bases its measurements on the heartbeat. If you have a weak or irregular heartbeat, the device may have difficulties determining your blood pressure.
- ☐ If the device detects a condition that is abnormal, it will stop the measurement and the error symbol will be displayed. See pages 7 and 8 for the description of symbols.
- ☐ This blood pressure monitor is intended for use by people 13 years and older. Consult with your physician before using this device on a child. Unattended child should not use this device.
- ☐ The automatic blood pressure monitor's performance may be affected by excessive temperature or humidity, or altitude.

6. The C.P.G. Function Switch

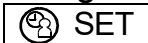- ❑ Refer to page 19 for the C.P.G. function that will indicate the proper angle so that the height of the cuff is the same level as your heart.
- 1. After step 6 in page 10, press the ◀ button to select either "on" or "off" concerning the C.P.G. function.
- 2. Press the  SET button to store the selection.
- 3. ❑ If you do not use my C.P.G. function, press  button to turn off. Proceed to "8. Measurement".
- ❑ If you use my C.P.G. function, proceed to "7. Selecting the C.P.G. and my C.P.G."

7. Selecting the C.P.G. and my C.P.G.

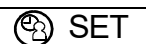- ❑ You can select an indicator either the C.P.G. function or my C.P.G. .
 - ❑ Preset a proper posture (wrist angle) in memory if you use the my C.P.G. .
 - 1. Adjust and keep the height of the blood pressure monitor to the same level as your heart using your wrist angle.
 - 2. Select an indicator using the ◀ button.
- Indicator The C.P.G. function is used. (my C.P.G. function : **OFF**)
- Indicator Data of my C.P.G. is deleted. Proceed to step 3.
- Indicator my C.P.G. function is **ON** and the current angle is stored when switched to the indicator  . Proceed to step 3.
3. Press the  button to turn off the device.
- Note: ❑ When removing the batteries, preset parameters (of clock, user and my C.P.G.) are reset.
- ❑ Select a user from user  and user  using the  SET button.

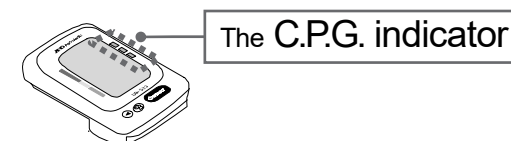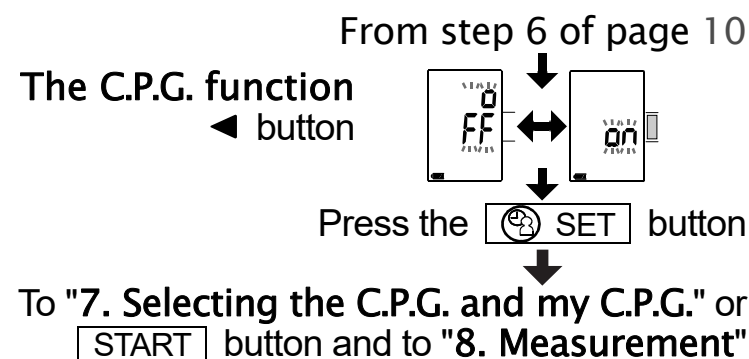


Step 1 [Adjust and keep the height of the device (wrist angle)]

HIGH (Orange light)

Correct Height (Blue light)

LOW (Orange light)

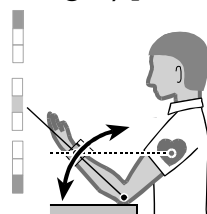

Step 2 [Select The C.P.G. or my C.P.G.]



8. Measurement

During the measurement, it is normal for users to feel tight.

9. After Measurement

While readings are displayed, if you press the START button to turn the device off, new readings are stored in memory.

While readings are displayed, if you press the ◀ button to turn the device off, new readings are not stored. Remove the cuff and the readings will be recorded.

Notes: The device is provided with an automatic power shut-off function which stores the current data in the memory and turns the device off automatically one minute after measurement. Allow at least three minutes between measurements on the same person.

Measurements

Note: The UB-533, once used, will provide an inflation appropriate to the user.

1. Wrap the cuff around your wrist. Sit comfortably with the cuff at the same level as your heart and relax.

2. Press the **START** button. All of the display segments are displayed.

3. Select a user from user 1 and user 2 using the **SET** button immediately. Adjust and keep the height of the cuff (with blue light) to the same level as your heart using the C.P.G. indicator.

Notes: If you do not use user selection, wait for the inflation for several seconds.
If you do not use the C.P.G. function, the C.P.G. indicator is not displayed.

4. Zero (0) is displayed blinking briefly. Then the display changes, as measurement begins. The cuff starts to inflate. It is normal for the users to feel tight. The measurement starts automatically when inflation starts, and the ♥ (heart mark) blinks.


Note: If you want to stop inflation at any time, press the **START** button again.

5. When the measurement is completed, the device displays readings (of the systolic and diastolic blood pressure readings, pulse rate, WHO classification, and IHB symbol and C.P.G. indicator). While readings are displayed, the date and time are displayed alternately. The cuff exhausts the remaining air and completely deflates automatically.

Note: If you do not want to store new readings in memory, press the ◀ button while readings are displayed.

6. Press the **START** button again to turn the device off. Remove the cuff.


Notes: The device is provided with an automatic power shut-off function.
Allow at least three minute between measurements on the same person.

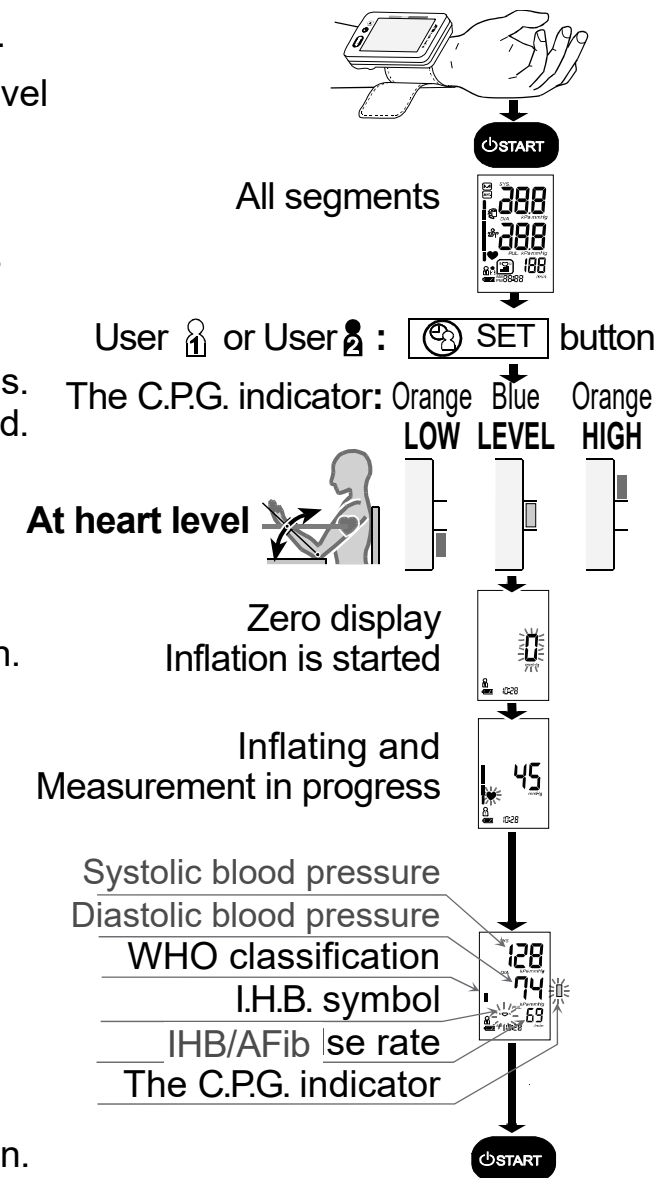


Recalling the Memory Data

Note: This device stores the last 60 measurements in memory.


1. Press the ◀ button when turning the device off.
The average of all measurements and the number of data are displayed.
If no data, "0" is displayed. Press the ◀ or [START] button to turn the device off.
2. Use the following buttons to display data (of number and measurement data).
 - Select a user from user 1 and user 2 using the [SET] button.
The device displays the average of all measurements and the number of data are displayed.
 - Pressing the ◀ button each time, the device displays as follows:
 - Average data of all AM (morning) measurements taken between 4:00 and 9:59. In the example, If no data, "--" is displayed.
 - Average data of all PM (evening) measurements taken between 18:00 and 1:59.
 - Data (of number and measurement data).
The device displays in order from most recent data. The date and time are displayed alternately while displaying the measurement data.
In the example: No.30 & data → No.29 & data → ... → No.01 & data.
3. If you press the ◀ button after oldest data is displayed, the device proceeds to step 1, the average of all measurements and the number of data are displayed.
4. Press the [START] button to turn the device off.
After one minute of non-operation, the device will be turned off automatically.

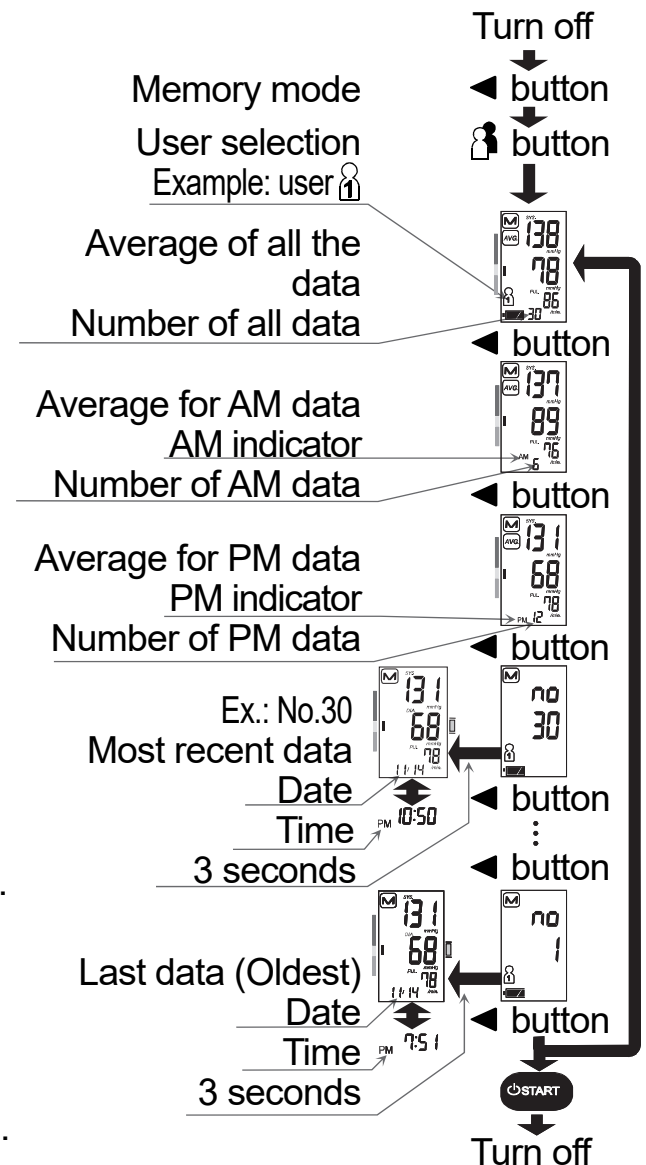

Deleting Data Stored in Memory

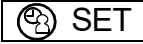1. Select a user from user 1 and user 2 using the  SET button. Turn off the device using the  button.
2. Press and hold the  button until turning off the device automatically. The device displays a user icon and the  mark, deletes data stored in memory while blinking the  mark and turns off automatically.

Note: This operation will delete specified user data stored in memory.
You cannot select which data to delete.

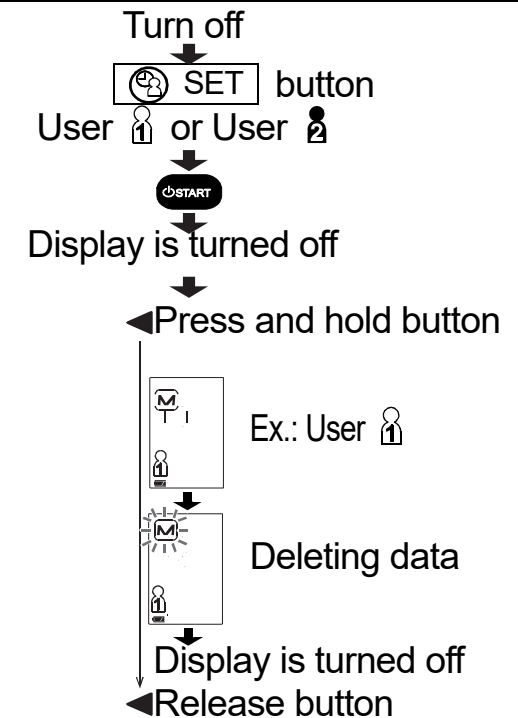

What Is the IHB/AFib Indicator?

When the monitor detects an irregular rhythm during the measurements, the IHB/AFib indicator will appear on the display with the measurement values.

Note: We recommend contacting your physician if you see this  IHB/AFib indicator frequently.

What Is the AFib?

The heart contracts due to electrical signals occurring in heart and sends blood through the body. Atrial fibrillation (AFib) occurs when the electrical signal in the atrium becomes confused and leads to disturbances in the pulse interval. AFib can cause blood to stagnate in the heart, which can easily create clots of blood, a cause of stroke and heart attack.

% IHB/AFib

%IHB/AFib is displayed as frequency of IHB detected.


IHB/AFib can detect not only noises such as physical movement but also an irregular heartbeat. Therefore, we recommend contacting your physician if %IHB/AFib level is high.

$$\%IHB/AFib = \frac{\left[\begin{array}{c} \text{Number of detected} \\ \text{IHB/AFib s in memory} \end{array} \right]}{\left[\text{Total number} \right]} \times 100 \%$$

Display of %IHB/AFib: %IHB/AFib is displayed when displaying average values.

%IHB/AFib is not displayed when the memory number is six or less.


Average value display

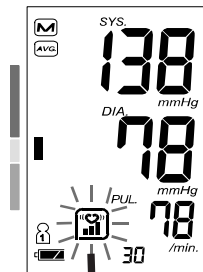

Level 0 %IHB/AFib=0	Level 1 %IHB/AFib=1 - 9	Level 2 %IHB/AFib=10 - 24	Level 3 %IHB/AFib=25 - 100
Not displayed			

Pressure Bar Indicator

The indicator monitors the progress of the pressure during the measurement.

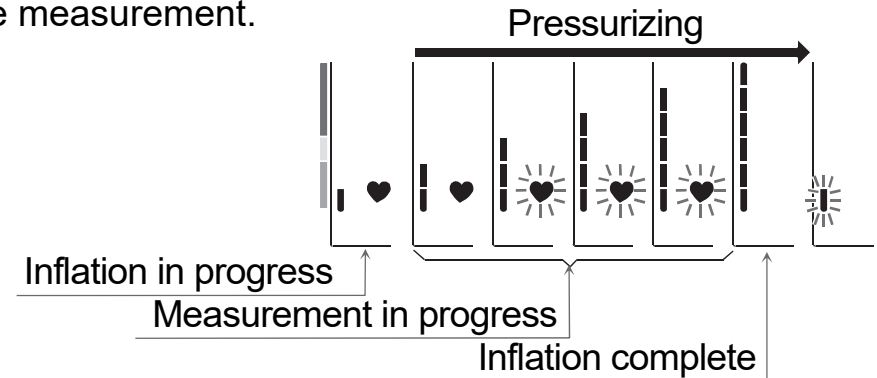


WHO Classification Indicator

Each six segments of the bar indicator correspond to the WHO blood pressure classification is described on 21 page.

Example

Moderate hypertension	Mild hypertension	High normal
SYS. 174 DIA. 102 PUL. 80 /min.	SYS. 147 DIA. 98 PUL. 84 /min.	SYS. 134 DIA. 87 PUL. 87 /min.

WHO Classification Indicator

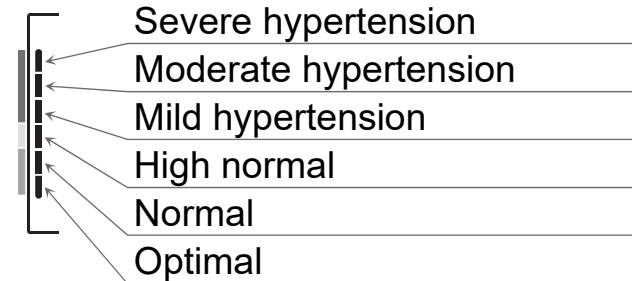


! : The indicator displays a segment, based on the current data, corresponding to the WHO classification.


The C.P.G. Indicator

❑ The C.P.G. Indicator

The C.P.G. (**C**orrect **P**osition **G**uidance) indicator is the function to inform a difference between the height (wrist angle) of the blood pressure monitor and your cardiac height in the correct posture (Example: sitting posture, height of table and chair, etc.) during the measurement. The indicator can be used to get more stable measurement condition.

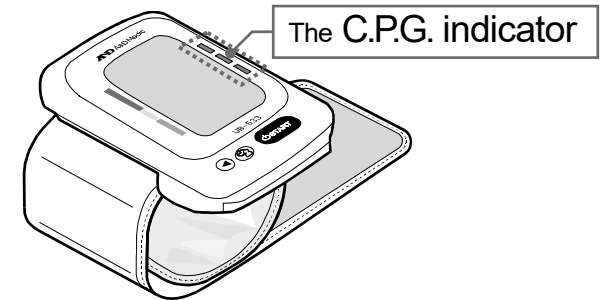

❑ The C.P.G. Indicator

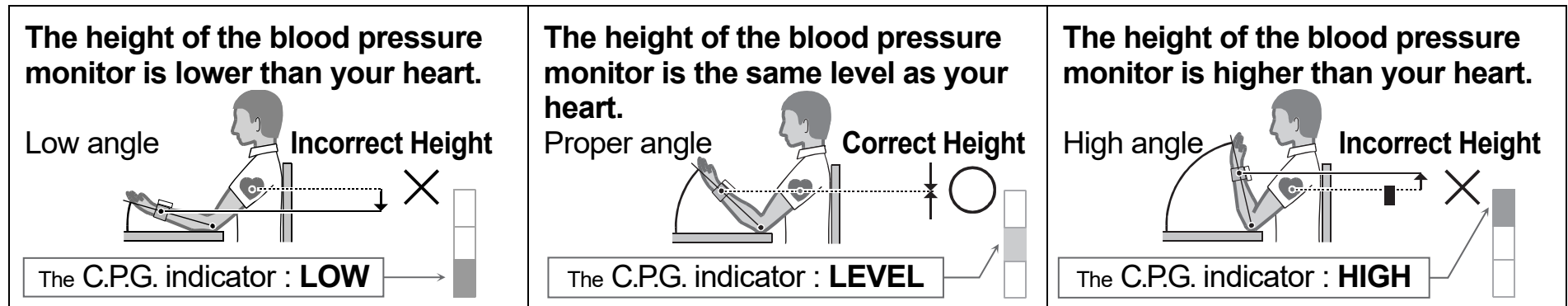


The position of the device is checked both before and after measurement. If both checks show a correct measurement position, the LEVEL indicator is lit (blue). For all other measurements, an indicator for LOW or HIGH measurement position will be lit (orange).

❑ How To Use My C.P.G.

The C.P.G. function can be used with proper posture (wrist angle) in the majority of measurements. If you need to change the posture to adjust the height so that the height of the blood pressure monitor is the same level as your cardiac height, you can use my C.P.G. function to store a personal posture. Preset your angle to my C.P.G. function before measurement.

- ❑ **Proper Posture**
Sit straight back on a chair and put your elbow on a table.

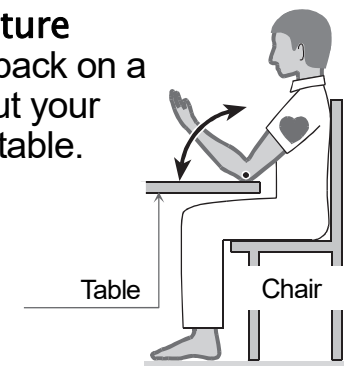

☐ **The Indicator During Measurement And Recalling Memory**

The C.P.G. indicator can be displayed at the beginning of measurement and is included in data stored in the memory. Adjust and keep the angle during the measurement.

About Blood Pressure

What is Blood Pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic blood pressure occurs when the heart contracts. Diastolic blood pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg). One's natural blood pressure is represented by the fundamental pressure, which is measured first thing in the morning while one is still at rest and before eating.

What is Hypertension and How is it Controlled?

Hypertension, an abnormally high arterial blood pressure, if left untreated, may cause many health problems including stroke and heart attack. Hypertension can be controlled by altering lifestyle, avoiding stress, and with medication under a doctor's supervision.

To prevent hypertension or keep it under control:

- ☐ Do not smoke
- ☐ Exercise regularly
- ☐ Reduce salt and fat intake
- ☐ Have regular physical checkups
- ☐ Maintain proper weight

Why Measure Blood Pressure at Home?

Blood pressure measured at a clinic or doctor's office may cause apprehension and may elevate the reading by 25 to 30 mmHg higher than when measured at home. Home measurement reduces the effects of outside influences on blood pressure readings, supplements the doctor's readings and provides a more accurate, complete blood pressure history.



WHO Blood Pressure Classification

Standards to assess high blood pressure, without regard to age, have been established by the World Health Organization (WHO), as shown in the chart at the right.

Blood Pressure Variations

An individual's blood pressure varies greatly on a daily and seasonal basis. It may vary by 30 to 50 mmHg due to various conditions during the day. In hypertensive individuals, variations are even more pronounced. Normally, the blood pressure rises when working or playing and falls to its lowest levels during sleep. So, do not be overly concerned by the results of one measurement.

Take measurements at the same time every day using the procedure described in this manual to get to know your normal blood pressure. Regular readings give a more comprehensive blood pressure history. Be sure to note the date and time when recording your blood pressure. Consult your doctor to interpret your blood pressure data.

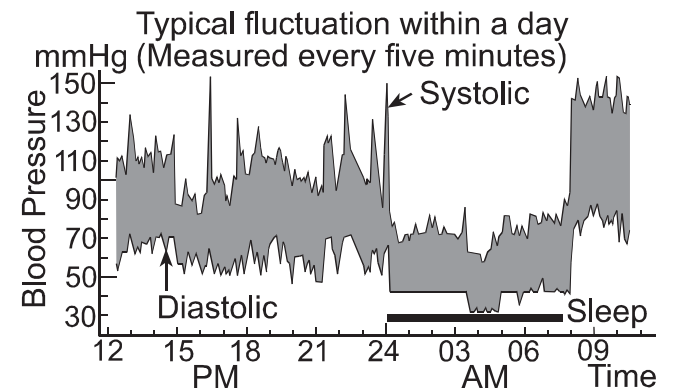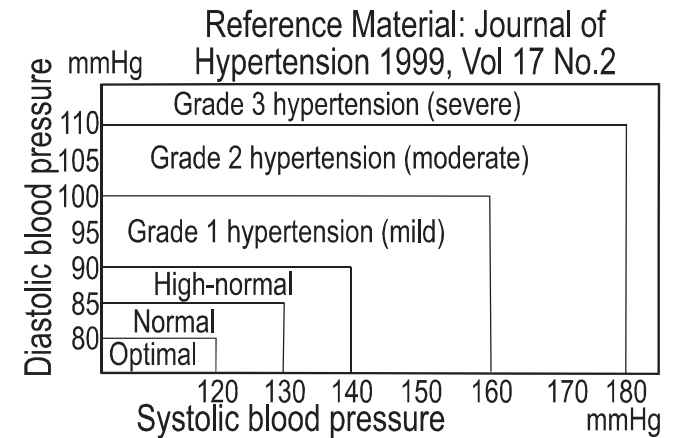

Troubleshooting

Problem	Possible Cause	Suggestion
Nothing appears in the display, even when the device is turned on.	Batteries are drained.	Replace all batteries with new ones.
	Battery terminals are not in the correct position.	Reinstall the batteries with negative and positive terminals matching those indicated in the battery compartment.
The cuff does not inflate.	Battery power is too low.  (LOW BATTERY mark) blinks. If the batteries are drained completely, the mark does not appear.	Replace all batteries with new ones.
The device does not measure. Readings are too high or too low.	The cuff is not applied properly.	Apply the cuff correctly.
	Movement is detected.	Make sure to remain very still and quiet during the measurement.
	The cuff position is not correct.	Sit comfortably and still. Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.
	_____	If you have a very weak or irregular heartbeat, the device may have difficulties in determining your blood pressure.
Other	The value is different from that measured at a clinic or doctor's office.	See the section "Why Measure Blood Pressure at Home?".
	_____	Remove the batteries. Put them back properly and try measurement again.

Note: If the suggestions described above do not solve the problem, contact the authorized dealer. Do not attempt to open or repair this product yourself, otherwise your warranty may be invalid.

Maintenance

Do not open the device. It uses delicate electrical components and an intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact the authorized dealer in your area or our customer service department. The A&D customer service will provide technical information, spare parts and units to authorized dealers.

The device is designed and manufactured for a long service life. However, it is generally recommended to have the device inspected every 2 years, to ensure proper functioning and accuracy. Please contact the authorized dealer in your area or A&D for maintenance.

Technical Data

Type	UB-533
Measurement method	Oscillometric measurement
Measurement range	Pressure: 0 – 299 mmHg Systolic pressure: 60 – 279 mmHg Diastolic pressure: 40 – 200 mmHg Pulse: 40 – 180 beats / minute
Measurement accuracy	Pressure: ± 3 mmHg Pulse: ± 5 %
Power supply	2 x 1.5 V alkaline batteries (LR03 or AAA)
Number of measurements	Approx. 200 measurements, when AAA alkaline batteries are used, with pressure value of 170 mmHg at room temperature of 23 °C.
Wrist circumference	13.5 – 21.5 cm
Classification	Internally powered ME equipment (Continuous operation mode)
Applied part	Cuff Type BF 

Useful life	Device: 5 years (when used six times a day)
Contents *2	1 Blood Pressure Monitor, 1 Instruction Manual, 2 Batteries, 1 Carrying case 1 EMD Technical Documentation
Clinical test	According to ISO81060-2 : 2013
EMD	IEC 60601-1-2: 2014+A1: 2020
Memory	Last 60 measurements each for user 1 and user 2.
Operating conditions	+10 to +40 °C / 15 to 85 %RH / 800 to 1060 hPa
Transport / storage conditions	-20 to +60 °C / 10 to 95 %RH / 700 to 1060 hPa
Dimensions	Approx. 56 [W] x 88 [H] x 21.5 [D] mm
Weight	Approx. 95 g, excluding the batteries
Ingress protection	IP22

*2: Confirm that all of the parts are included to ensure that the medical device is ready to perform safely and as intended.

Note: Specifications are subject to change for improvement without prior notice.

IP classification is the degrees of protection provided by enclosures in accordance with IEC 60529. This device is protected against solid foreign objects with a diameter of 12 mm or greater, such as a finger. The device is protected against water drops falling vertically when the device is tilted at any angle up to 15°.

MEMO

[illegible]

**A&D Company, Limited**

1-243 Asahi, Kitamoto-shi, Saitama-ken 364-8585 Japan
Telephone: [81] (48) 593-1111 Fax: [81] (48) 593-1119

**Emergo Europe B.V.**

Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

**A&D INSTRUMENTS LIMITED**

Unit 24/26 Blacklands Way, Abingdon Business Park, Abingdon, Oxfordshire, OX14 1DY, United Kingdom
Telephone: [44] (1235) 550420 Fax: [44] (1235) 550485

A&D Engineering, Inc.

4622 Runway Boulevard, Ann Arbor, MI 48108 USA
Telephone: [1] (888) 726-9966

A&D AUSTRALASIA PTY LTD

32 Dew Street, Thebarton, South Australia 5031, AUSTRALIA
Telephone: [61] (8) 8301-8100 Fax: [61] (8) 8352-7409

ООО A&D RUS

ООО "ЭЙ энд ДИ РУС"

121357, Российская Федерация, г.Москва, ул. Берейская, дом 17
(Business-Center "Vereyskaya Plaza-2" 121357, Russian Federation, Moscow, Vereyskaya Street 17)
тел.: [7] (495) 937-33-44 факс: [7] (495) 937-55-66

A&D Technology Trading(Shanghai) Co. Ltd 爱安德技研贸易(上海)有限公司

中国 上海市自由贸易试验区浦东南路855号世界广场32楼C,D室 邮编200120
(32CD, World Plaza, No.855 South Pudong Road, China (Shanghai) Pilot Free Trade Zone, 200120, China)
电话: [86] (21) 3393-2340 传真: [86] (21) 3393-2347

A&D INSTRUMENTS INDIA PRIVATE LIMITED ऐंडी इन्स्ट्रुमेंट्स इण्डिया प्रा० लिमिटेड

509, उद्योग विहार, फेस -5, गुडगांव - 122016, हरियाणा, भारत
(509, Udyog Vihar, Phase-V, Gurgaon - 122 016, Haryana, India)
फोन : 91-124-4715555 फैक्स : 91-124-4715599

